Ardo care compresses

Ardo Care Compresses

Indication and area of use: Ardo Care Compresses provide breastfeeding mothers with natural care and support for the treatment of sore, irritated or painful nipples. The gel on the non-woven fabric compress supports the natural regeneration of the skin.

Dosage and application:

1. Thoroughly wash your hands with soap and water.
2. Place an Ardo Care Compress with the gel side on the nipple. Place a normal nursing pad.
3. Leave the compress in place for as long as needed, but for at least 10 minutes, better for 30 minutes up to a maximum of one hour. The residue of the gel on the breast does not need to be wiped off before breastfeeding, it is harmless for baby.
4. Repeat the treatment at least twice a day until the irritation has disappeared. Use a compress for one application only.

Type of administration: For local application on the nipples.

Precautions: There are no precautionary measures needed.

Side effects/interactions: None are known.

Contraindications: Do not apply with known hypersensitivity to any of the ingredients.

Note for users: According to Regulation (EU) 2017/745 on medical devices, any serious incidents that occur in connection with the product must be reported to the manufacturer and the competent authorities of the Member State in which the user resides.

Dosage form and content: 12 individually packed compresses

Properties and pharmaceutical form: Odourless, colourless gel with a natural basis, applied to a round non-woven fabric compress that is coated on one side. The compresses do not contain any medically active ingredients.

Storage and shelf life: Store at +5 to +30 °C and protect from direct light. The expiry date is printed on the sachet and the pack. Do not use after the expiry date.

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